



Global Health  
Technology


# ERLAND CARE

Reaching new heights in  
Ostomy and Wound Care



# Erland Care Protective Skin

Is a multifunctional wound dressing designed to support optimal wound healing. Its superabsorbent core effectively binds exudate, while its activated fibers regulate moisture levels to create an ideal wound environment. The dressing is a non-sterile Class 1 device.

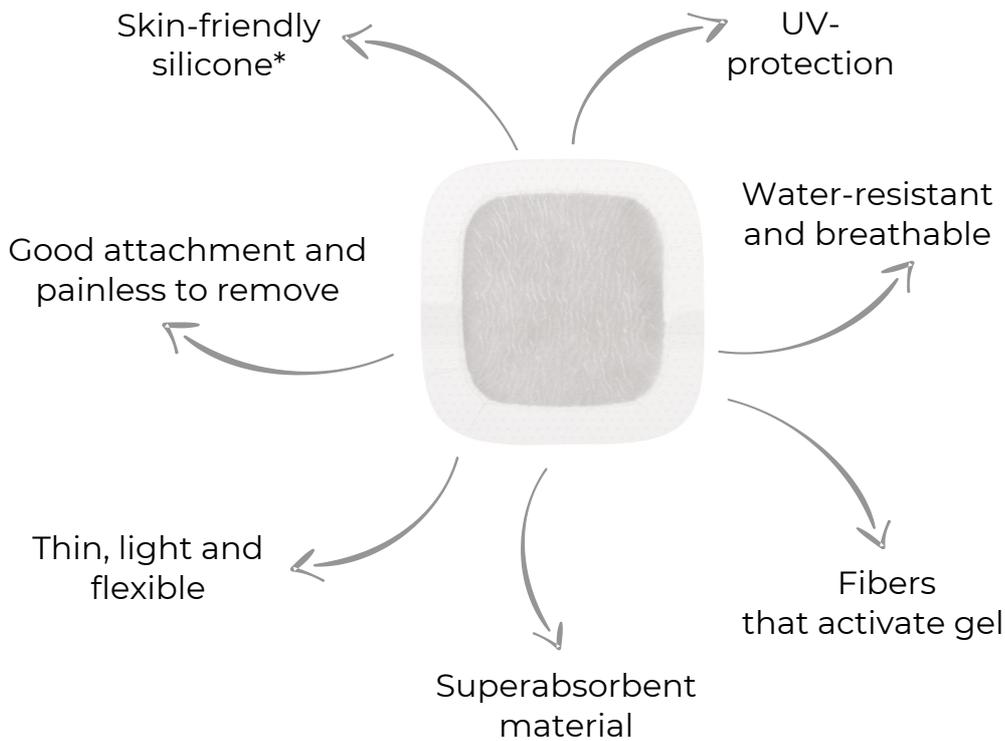

## Erland Care Protective Skin

Was developed for people with fragile and sensitive skin.

Global Health Technology is collaboration partner with DEBRA Norway\*.

### Areas of use:

- ✓ Dry and exuding wounds
- ✓ Arterial wounds
- ✓ Venous leg ulcers
- ✓ Pressure ulcers
- ✓ Diabetic wounds
- ✓ Surgical wounds and scars
- ✓ First- and second-degree burns
- ✓ Traumatic wounds
- ✓ Peristomal wounds
- ✓ Covering of continent stomas
- ✓ Epidermolysis Bullosa\*


### Available in 6 sizes:

Dressing sizes in (cm)	Super-absorbent	Silicone border	Thickness
<b>5 x 5</b>	2 x 2	1,5	0,2
<b>7,5 x 7,5</b>	5 x 5	1,25	0,2
<b>10 x 10</b>	7,5 x 7,5	1,25	0,2
<b>15 x 15</b>	13,5 x 13,5	0,75	0,2
<b>16,2 x 18,4</b>	13,2 x 15,4	1,5	0,2
<b>5 x 25</b>	2 x 22	1,5	0,2

\* DEBRA Norway is a nationwide advocacy organization for individuals with Epidermolysis Bullosa (EB).

# Erland Care Protective Skin 5x25 cm

This 5 x 25 cm silicone dressing was specially designed for use after surgery, like c-section or heart bypass surgery. It can be used directly on wounds and scars.

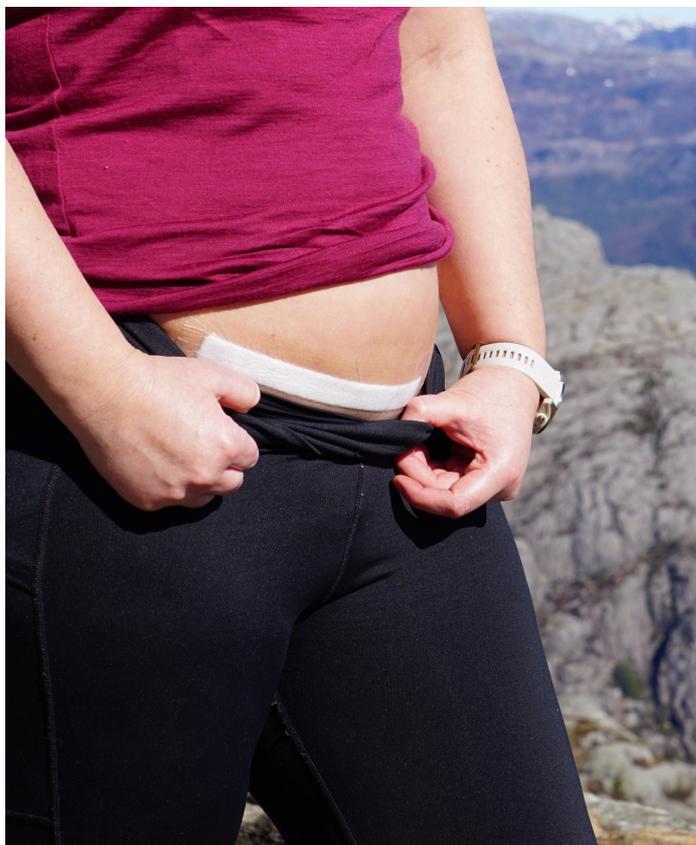

## Suitable postoperative



Breast augmentation



Over joints like knee, shoulder, elbow and hip



Abdominal, midline and transverse incisions



Thigh and lower body lift

## Benefits

- ✓ Gentle on the skin, light and very flexible
- ✓ Easy and painless to remove
- ✓ Suitable to wear when swimming or bathing
- ✓ Provides UV protection

Each dressing can be used for  7 days.



## Reduces the visibility of scars and minimizes the risk of infection

Clinical studies indicate that applying a silicone dressing for six months post-surgery significantly reduces scar prominence and the likelihood of infection.\*

\*Ying-Sheng Lin, Advances in Skin and Wound Care, June 2020



# Erland Care Protective Skin: Peristomal Wounds

## Treatment of peristomal wounds/ moisture associated skin damage (MASD)


A review of 37 published professional studies in the area provides updated knowledge and new recommendations on the treatment of moisture-associated skin damage. In the article "Management of Moisture-Associated Skin Damage: A Scoping Review published in Advanced Wound Care in 2017", the use of absorbent dressing is set as a first choice in treatment.

### Recommended treatment for MASD:


1. Cleanse the skin gently using a neutral, non-irritating soap to avoid further damage.
2. Apply an absorbent dressing to manage exudate. The dressing should: Prevent the formation of new wounds, support the healing of existing wounds, be capable of absorbing high volumes of fluid to maintain an optimal moisture balance.

### How to use: Protective Skin 15x15 cm or 16,2x18,4 cm under baseplate


Cut the hole slightly larger than the stoma so that the dressing does not touch the stoma.

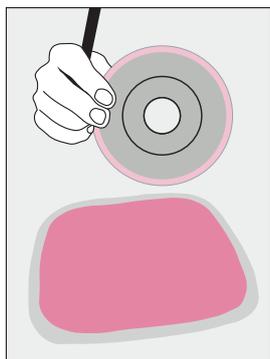

1. Take out a pre-cut base-plate that has been adjusted to your stoma size.

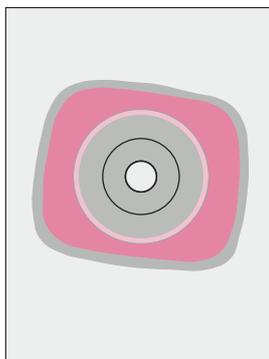


2. Place the baseplate over the Erland Care Protective Skin.

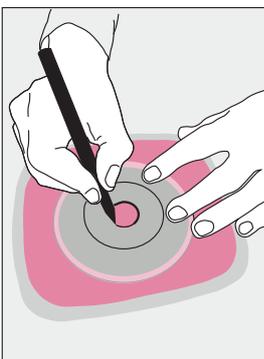


3. Draw a mark of the hole for your stoma.

“Up to **80%** of all people with a stoma experience peristomal skin complications, most of which are related to a poor seal around the pouch.”


Colwell et al, 2019

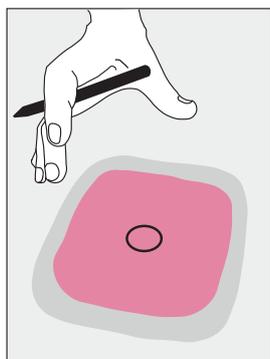

4. Remove the base-plate, after the hole is marked.

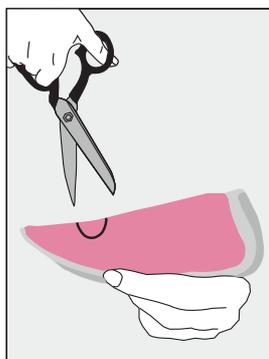

5. Fold the Erland Care Protective Skin in half and cut around the marked area.

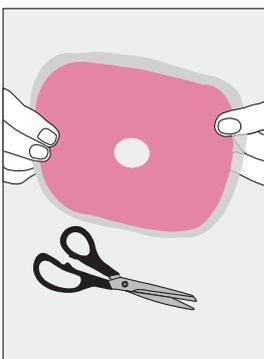

6. The Erland Care Protective Skin is ready to be placed on the skin.



Any one- or two-piece system can be attached directly on top of the dressing.



# Erland Care Stoma Pad

## The patented and skin-friendly 360° silicone flange extender

Erland Care Stoma Pad ensures safe and stable attachment of bag and baseplate, which in turn reduces risk of leakage under the baseplate. It absorbs leakage and seals it in the superabsorbent, reducing odour and preventing liquids from getting on the skin, clothes, bedding, etc.

Perforated silicone as an adhesive provides the skin with good ventilation and makes it easy and painless to remove after use.


The product can safely be used in water as it is water-resistant.

Fits baseplate sizes up to Ø 10 cm and is suitable for one- and two-piece systems.




## How to use: Example one-piece stoma system


The Stoma Pad will always be put **on top** of the baseplate like a flange extender.

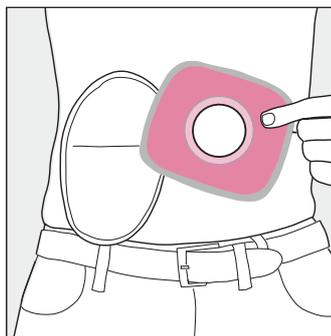


**1.** Put on the ostomy bag as usual. Ensure the baseplate is attached well. Take the Stoma Pad out of the package.

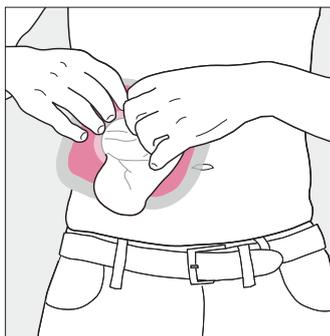


**2.** Hold the Stoma Pad with the adhesive side against the skin, then pull the stoma bag through the hole.

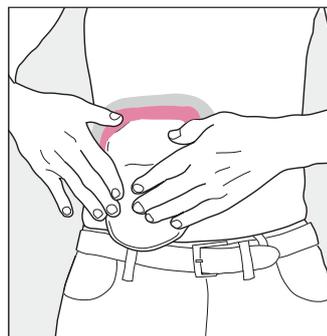


**3.** When the entire stoma bag has been pulled through the hole, straighten out the bag.

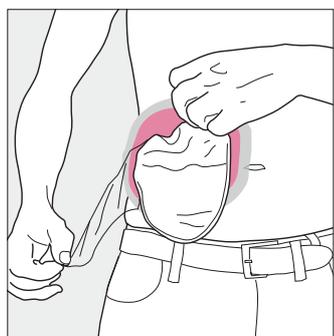

**4.** Remove one of the back liners and attach the Stoma Pad on the skin. Repeat with the other back liner.

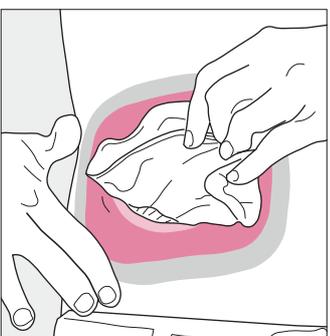

**5.** Smooth out the Stoma Pad and check that there are no openings on the sides or air pockets.

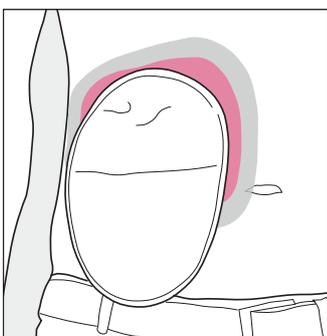

**6.** The Stoma Pad is ready to be used.

### Two-piece system:

1. Attach the baseplate as usual to the skin.
2. Attach the Stoma Pad on top of the baseplate, by following steps 4 & 5.
3. Attach the stoma bag to the baseplate.



# Customer Reviews and Recognition



2025

“Innovation of the Year”

Wound Congress DEWU,  
Germany



2023

“Newcomer of the Year”

Banda stores,  
Norway



2022

“Social Care Premier  
Supplier Healthcare”

Social Care,  
United Kingdom

★★★★★ **District Nurses loved it**

Review from the United Kingdom on August 24, 2023

Size: 16.2 x 18.4cm | **Verified purchase**

My wife suffers from an auto immune condition which attacks the skin leaving her with painful open sores which needs dressings. In between visits from the District Nurses I ran out of the dressings which they had left me so I ordered these. They were perfect from the job, and the District Nurses preferred these to the ones that they NHS ordered for them.

★★★★★ **Pleasant on the skin.**

Review from Germany from May 30, 2024

Size: 7.5 x 7.5cm | **Verified purchase**

I ordered this wound dressing for my father, who occasionally has ear problems (he has very sensitive skin). The wound seemed to heal better than usual with it and came off easily, so I ordered another pack.

★★★★★ **Fast healing**

Review from Germany from April 24, 2025

Size: 7.5 x 7.5cm | **Verified purchase**

My husband was discharged from the hospital with five bedsores, and with these wonderful plasters, all five of them healed. I'm thrilled and can only say positive things.

★★★★★ **Good wound dressing.**

Review from Germany from February 1, 2025

Size: 7.5 x 7.5cm | **Verified purchase**

A wound dressing with enormous adhesive strength.

★★★★★ **Top and affordable**

Review from Germany from August 10, 2025

Size: 7.5 x 7.5cm | **Verified purchase**

Holds firmly for days

Product name		Reference number
Erland Care Stoma Pad (box of 10)		MED PAD 0118
Erland Care Protective Skin (box of 10)	5x5 cm	MED PAD 0419
	7,5x7,5 cm	MED PAD 0319
	10x10 cm	MED PAD 0219
	15x15 cm	MED PAD 0119
	16x18 cm	MED PAD 0519
	5x25 cm	MED PAD 0622

## Contact us

Service phone: +47 51 22 55 70

E-mail: [post@erlandcare.no](mailto:post@erlandcare.no)



For more information and the instructions for use, please visit

[www.globalhealthtechnology.no](http://www.globalhealthtechnology.no)

## **GLOBAL HEALTH TECHNOLOGY**

is a Norwegian company that develops effective and patient-centered solutions for people with stomas and wounds to help them regain their quality of life.

Global Health Technology AS  
Løkkeveien 107-111  
4007 Stavanger  
NORWAY

